


Programming Plum 360 Multi-Step Infusion

Blood IVIG 10% Generic with Callback

Training videos available through the Smart Pump Program

- <https://www.icumed.com/plum-360-with-mednet-training>
- Can also be found by scanning the QR code

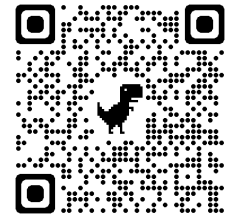

Selecting Dose

- Select Dose 1 for any first-time IVIG infusions
- Select Dose 2+ for subsequent IVIG infusions

Multistep Infusion on Line A

- Select Multistep before entering any values
- **Select Add Callback**
- Enter Adjusted Body Weight of Patient
- Arrow down to 1
- Enter Rate from infusion rate table range of the Adjusted Body Weight
- Enter duration of 30 minutes — note VTBI is auto calculated
- Arrow down to 2
- Enter Rate from infusion rate table range of the Adjusted Body Weight
- Enter Duration of 30 minutes
- Arrow down to 3
- Enter duration of 30 minutes
- Press steps 4–10
- Enter Rate from infusion table
- Calculate the volume infused in steps 1-3 subtract from total volume to calculate the VTBI
- Press start—Review steps to confirm Multistep
- Press start to begin infusion



Healthy People, Healthy Saskatchewan

The Saskatchewan Health Authority works in the spirit of truth and reconciliation, acknowledging Saskatchewan as the traditional territory of First Nations and Métis People.



saskhealthauthority.ca